



**B. Cool – B. Braun**



**Askina<sup>®</sup> Skin Freeze**

Cryotherapeutic device for treating benign skin lesions

# Askina® Skin Freeze

Cryotherapeutic device for treating benign skin lesions

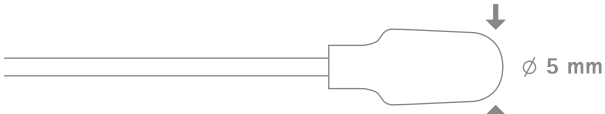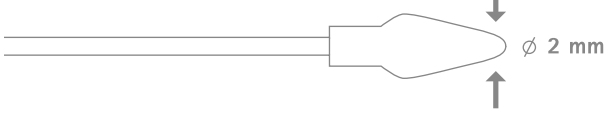

Askina® Skin Freeze		REF
Kit for standard therapy	 A diagram showing the tip of the 5 mm Askina Skin Freeze probe. It is a rounded, bulbous shape with a diameter of 5 mm, indicated by a double-headed arrow and the text "ø 5 mm".	9380809
Kit for small lesions	 A diagram showing the tip of the 2 mm Askina Skin Freeze probe. It is a pointed, conical shape with a diameter of 2 mm, indicated by a double-headed arrow and the text "ø 2 mm".	9380701

## KIT CONTENT


- 150 mL can filled with
  - 95% dimethyl ether
  - 3% isobutane
  - 2% propane(under pressure as liquid)
- Mounted applicator holder
- Box with 50 applicators (5 mm or 2 mm tip)




## HOW TO USE



1 Insert the applicator into the green applicator reservoir, spray for approximately 5 seconds

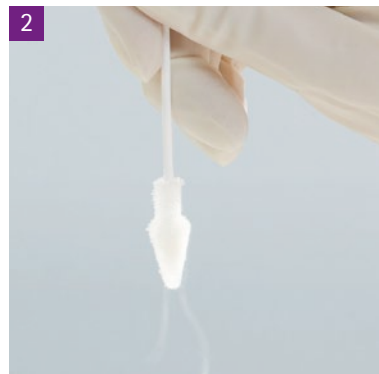

2 Take out the applicator and wait for 15 seconds

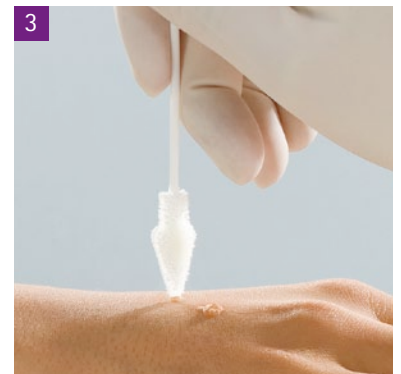

3 Freeze the wart according to the type of lesion (15 – 40 seconds of freezing treatment)

