

Optimize recovery for cats in your practice.


The MPS-Protective Front Leg Single Sleeve can be used for protection on the front leg during hotspots, wounds, bandages, and skin problems. The sleeve can even be useful in covering ointments to prevent the cat from licking the medicine. Expedite the healing process of each cat with the Single Sleeve, made of high-quality cotton and 4-way-stretch Invista Lycra® for the optimal comfort and care.

Why the MPS-Protective Single Sleeve sets the standard in pet recovery

- ✓ Can protect IV lines.
- ✓ To be used on the left or right front leg - Interchangeable.
- ✓ Easily adjustable for the perfect fit.
- ✓ Easy and quick check-ups.
- ✓ Available in 3 sizes.
- ✓ Helps reduce stress, and the gentle pressure may have a calming effect on some cats.



Sizing guide of the MPS-Protective Single Sleeve Cat

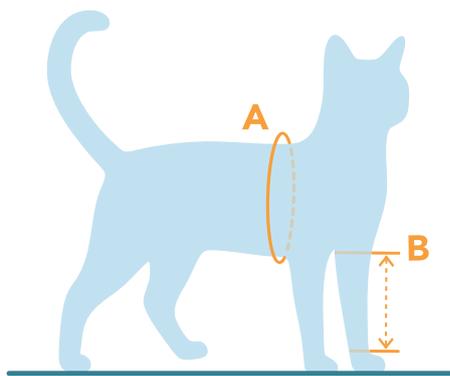

Size	Chest (A)		Length Leg (B)	
	cm	in	cm	in
2XS	30 - 42 cm	11.8 - 16.5 in	8 - 13 cm	3.1 - 5.1 in
XS	32 - 44 cm	12.6 - 17.3 in	10 - 15 cm	3.9 - 5.9 in
S	37 - 45 cm	14.6 - 17.7 in	12 - 17 cm	4.7 - 6.7 in

Size guide

Measure the chest circumference around the widest part of the **Chest (A)**, usually right behind the front legs. Measure the **Length of the Leg (B)** on the inside of the leg from the armpit to ankle.



How to put on the Protective Front Leg Single Sleeve

Putting the sleeve on:

Roll up the sleeve for easier access. Pull the recovering leg through the sleeve. Pull the other leg through the opening on the other side of the sleeve.

Securing the sleeve:

Secure the flaps around the chest by using the press studs and Velcro®. It is important to use at least two rows of press studs. Take a good look around the legs and chest area to see if there is enough room to move and breathe!

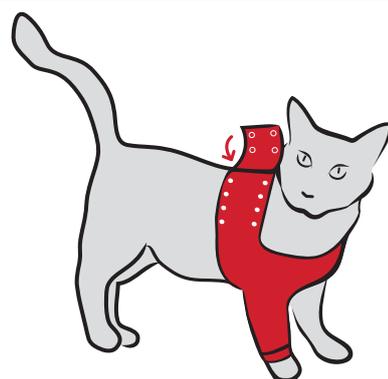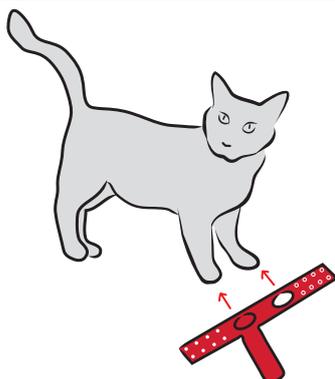

Features of the MPS-Protective Front Leg Single Sleeve Cat.



1. Press studs for a good fixation

The closure between the shoulder blades helps keep the Recovery Sleeve securely in place, even for active cats. The Velcro and press studs allow easy access for the owner or veterinarian to check the wound or remove the sleeve when needed.



2. Stretch bands

A soft, elastic and protective stretch-band around the chest ensures that the Recovery Single Sleeve won't chafe the leg and chest of the cat.



3. Soft long sleeve

A soft stretch band at the bottom of the sleeve ensures that the sleeve fits more snugly around the leg. Sleeves that are too long can be rolled up.



4. Interchangeable

The MPS-Protective Front Leg Single Sleeve has a symmetrical design which makes it applicable on the left and right leg. One leg goes into the sleeve and the other goes through the opening on the other side.

Washing instructions



- Always wash the sleeve inside out
- Fasten all press studs and Velcro® before washing
- Use a laundry bag

The washing, maintenance and handling of delivered items always remains the responsibility of the customer.

TIP! To ensure the cat remains protected, two sleeves are often purchased to ensure proper hygiene and protection in between washings. Especially when a sleeve is needed for more than one week, a backup is recommended.

The MPS-Protective Single Sleeve cat is available in 2 colors:

MPS-Red

MPS-Leopard

