


# BD Vacutainer® PrecisionGlide™ Multiple Sample Blood Collection Needles

## Intended purpose


The Multiple Sample Blood Collection Needle is intended for medical purposes to use with vacuum tubes for venous blood collection from a venous puncture. It is single-use and shall be disposed of into an approved medical waste container after use.

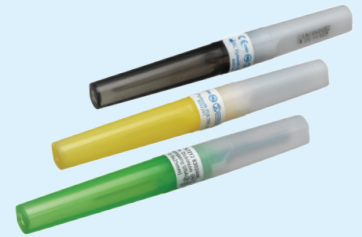

## How to use

Follow your facility's policies and procedures for venous blood collection. General guidance provided below.

- 1 Inspect** label and cap for any signs of damage. Do not use if the package is damaged.
- 2 Rotate** both caps to break the label, and remove the NP (Non-Patient) cap.
- 3 Screw** the NP Needle End to the Luer fitting of the Needle Holder.
- 4 Remove** the IV (Intravenous) Needle Cap, perform the venous puncture immediately.
- 5 Insert** the Blood Collection Tube into the Needle Holder to puncture the NP Needle End to collect the blood sample.

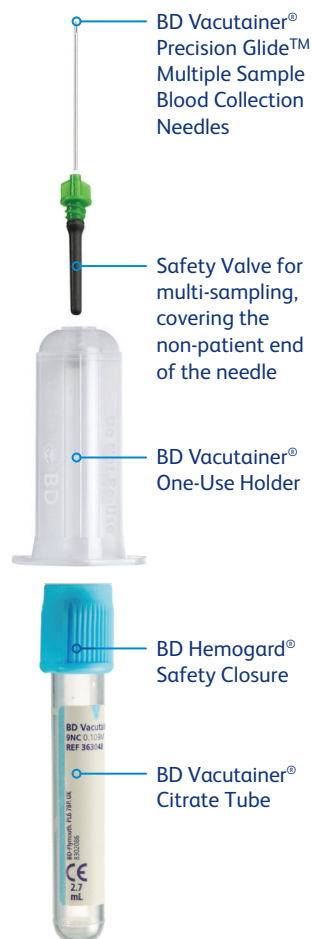

## Ordering information

### BD Vacutainer® PrecisionGlide™ Multiple Sample Blood Collection Needles

SKU	Description	Box/case	Cap colour
360220	22G (0.7 mm) - 25 mm	100/1,000	●
360221	22G (0.7 mm) - 38 mm	100/1,000	●
360222	21G (0.8 mm) - 25 mm	100/1,000	●
360223	21G (0.8 mm) - 38 mm	100/1,000	●
360225	20G (0.9 mm) - 38 mm	100/1,000	●
360227	18G (1.2 mm) - 38 mm	100/1,000	●

### BD Vacutainer® One-Use Holder

SKU	Description	Box/case
364815	BD Vacutainer® One-Use Holder is made of plastic and is designed to be part of the BD Vacutainer® System, compatible with tubes of 13 mm and 16 mm diameter	250/1,000

We recommend reviewing the Instructions for use (IFU) for detailed guidance and important information.

**CE**  
0123 CE0123 applies to BD Vacutainer® Precision Glide™ Multiple Sample Blood Collection Needles.

**CE**  
CE applies to BD Vacutainer® One-Use Holder and BD Vacutainer® Citrate Tube.

BD Switzerland Sarl, Terre Bonne Park – A4, Route de Crassier 17, 1262 Eysins, Switzerland

[bd.com](http://bd.com)

BD, the BD logo, PrecisionGlide and Vacutainer are trademarks of Becton, Dickinson and Company or its affiliates.  
© 2026 BD. All rights reserved. BD-165538 (11/25)

