

# TONO-PEN VET™

Veterinary Applanation Tonometer

The veterinarian's choice for rapid and accurate IOP measurements.





With over 40 breeds of dogs and cats possessing the predisposition to glaucoma, the Reichert **TONO-PEN VET™** Tonometer provides simplicity for monitoring this condition. This easy-to-use, hand-held instrument provides IOP readings for all Veterinary care. TONO-PEN VET has a lightweight, ergonomic design with advanced electronic measurement technology that enables operators to take fast and accurate IOP measurements with minimal training. The long-life OCU-CEL™ VET batteries operate for approximately 1,200 exams, ensuring that TONO-PEN VET will be ready to use when needed. Sanitized OCU-FILM® + Tip Covers are used to minimize the risk of cross-contamination between uses.

#### **Small and Lightweight**

Weighing only 2.25 ounces, TONO-PEN VET is extremely portable, making it easy to take IOP measurements anywhere, in any position.

## **Advanced Technology for Accuracy** and Repeatability

TONO-PEN VET utilizes micro strain gauge technology and a 1.0 mm diameter transducer tip. Leading Edge™ recognition software ensures that measurements are taken only at applanation, while ScanLock™ tracking software rapidly scans the electronic measurement data at a rate of 500 samples per second. This advanced microprocessor technology produces objective IOP measurements. The average of four independent readings, combined with a statistical confidence index, ensures accurate, repeatable, and reliable tonometry results.

#### Easy-to-Use

TONO-PEN VET functions in any orientation making it ideal for all of your measurements. The device activates with the push of a button and results are displayed clearly on the easy-to-read LCD screen. All will appreciate the quick and gentle measurement process. TONO-PEN VET delivers consistent results when operated by general veterinary practitioners, veterinary ophthalmologists, and other veterinary medical personnel.

## **Clinically Trusted**

Proudly Made in the USA.

For more information visit us online at

With the knowledge that IOP should be measured in all red eyes for which the cause is not immediately obvious, veterinarians worldwide have made TONO-PEN VET a trusted part of their practice for over two decades. The proven reliability of the device allows operators to take IOP measurements confidently anytime, anywhere, and in any position.

# **Catalog Numbers**

230635V TONO-PEN VET Tonometer Starter Pack (Contains TONO-PEN VET, Carry Case, 1 box of OCU-FILM + Tip Covers, 1 Package of OCU-CEL VET Batteries, Manual) 230651 OCU-FILM + Tip Covers - Box of 150 (Sanitized in Individual Packaging)

230652 OCU-FILM + Tip Covers - Bulk Pack of 200 230653 OCU-FILM + Tip Covers - Bulk Pack of 600 230570 OCU-CEL VET Batteries - 4 per package

## **Specifications**

Dimensions	7 1/4" x 1" x 7/8" (18.42 x 2.54 x 2.22 cm)
Weight	2.25 oz (64 g)
Transducer	Micro Strain Gauge
Measurement Range	5-80 mmHg
Power Source	2 OCU-CEL VET Batteries







Reichert, Inc.

3362 Walden Avenue, Depew, NY 14043 Toll Free: 888-849-8955 • Tel: (716) 686-4500 Fax: (716) 686-4545 • www.reichert.com

www.tonopen.com or call 1-800-TONO-PEN (1-800-866-6736)