

Detorsion with „one rope“ method



Detorsion „two-ropes“-/two persons-method



Put on ropes, fix them to the GYN-stick extravaginally; tighten the GYN-stick, it will automatically approach the calfs legs.
Fix the ropes outside on the distal hole of the GYN-stick, ready to turn = action.



Insert the metal bar, while vet is monitoring „intracorporally“ the process and farmer is „turning“ the wheel.
Both persons have engouh space to act in contrast to common obstetric devices, which are simply to short;
GYN-stick is also a very safe instrument to avoid ruptures of the uterus or broken legs of the calf.

Correction of displaced extremities



Turn the fine rope with the metal spoon around the proximal end of the displaced leg. Close the circle extravaginally on the end of the GYN-stick; direct this loop in distal direction of the leg in order to get closer to the assistant and for further actions serving the calving process.

