


BD PhaSeal™ Optima instructions for preparing an intravenous (IV) container using the infusion adapter (C70-O)

Always follow proper aseptic techniques and local guidelines for safe handling of drugs.

To prepare an IV container using the infusion adapter (C70-O), please review the following steps:

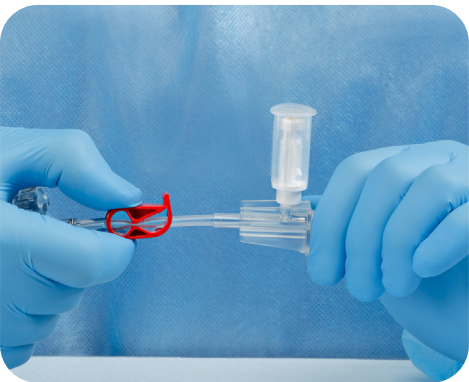

1 Close clamp.



2 Remove cap from spike. Push the spike fully into the IV bag.



3 Twist off cap from the lower spike port and insert into spike port.



4 Slide the injector straight onto the connection interface until the two pieces snap together.

Note:

- The BD PhaSeal™ Optima connector may be used for up to 10 penetrations.
- Ensure all luer lock connections are securely tightened.
- Use the required and accepted IV practice.

Please consult product labels and inserts for instructions for use, indications, contraindications, warnings and precautions.



5 Push drug into IV bag. Repeat steps 4 and 5 for additional syringe units.



6 Pull the injector straight back until the two pieces are disconnected.



7 Remove spike port cap. Hold bag spike port and insert IV set into the C70-O infusion adapter with IV container in horizontal position.



8 Connect and administer to patient as per institutional policy.



9 Dispose without disconnecting the BD PhaSeal™ Optima components.



Becton Dickinson UK Limited, 1030 Eskdale Road, Winnersh Triangle, Wokingham RG41 5TS

bd.com/en-uk

