



How to use the surgical clipper



1

Attaching blades

- Align the mark on the blade to the top line on the clipper.
- Slide the blade down until it aligns with the lower line on the clipper.
- A 'click' is heard.



2

Holding the clipper


- Hold the clipper like a pen.
- The non-slip handle side grips allow for full control of the clipper and one-handed manoeuvrability.



3

Clipping on wet or dry skin

- Make sure the clipper is in the 'ON' position.
- Use short, gentle strokes to remove the hair.
- For best results, the direction of the movement while clipping should go against the hair growth.

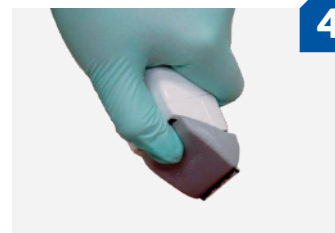

4

Removing blades

- Make sure the clipper is in the 'OFF' position.
- Place your thumb on the textured indentation on the back of the blade.
- Push until the blade pops off.



5

Cleaning and disinfecting

- Wipe down the handset with non-corrosive universal wipes between each patient use.
- Clean it by rinsing with water as soon as possible after each use.



6

Storing the clipper

- Make sure the clipper is in the 'OFF' position.
- Place it onto the charging station.

For more information on this product, please contact your Medline account manager or visit our website: uk.medline.eu