

	Test norm	Test type	Organism	Log reduction required	Log reduction achieved	Laboratory	Clean or dirty	Contact time	LC Std 400-200 PPM	LC Pluss 650-400 PPM
Sporicidal	EN 13697:2001 (Phase 2, Step 2)	Quantitative surface test	<i>Bacillus subtilis</i> ATCC 6633	4 log ₁₀	>4 log ₁₀	Dr. Brill + Partner GMBH Laboratory, Germany	Clean	2 min		✓
	EN 13697:2001 (Phase 2, Step 2)	Quantitative surface test	<i>Clostridium difficile</i> ¹ UK 027	>3 log ₁₀	>3 log ₁₀		Clean	2 min		✓
	EN 13704:2002 (Phase 2, Step 1)	Quantitative suspension test	<i>Clostridium difficile</i> ¹ UK 027	3 log ₁₀	>3 log ₁₀		Clean	5 min	✓	✓
					>3.3 log ₁₀				✓	✓
			<i>Clostridium difficile</i> ² UK 027		>6.1-7.5 log ₁₀	Clean	5-10 min	✓	✓	
	EN 17126:2018 (Phase 2, Step 1)	Quantitative suspension test	<i>Bacillus subtilis</i>	>4 log ₁₀	>5.02 log ₁₀	MSL Sloution, UK	Clean	2-5 min	✓	✓
			<i>Bacillus cereus</i>		>5.21 log ₁₀				✓	✓
			<i>Clostridium difficile</i>		>5.21 log ₁₀				✓	✓
	EN 17126:2018 (Phase 2, Step 1)	Quantitative suspension test	<i>Bacillus subtilis</i>	>4 log ₁₀	>4.08 log ₁₀	MSL Sloution, UK	Dirty	2 min	✓	✓
			<i>Bacillus cereus</i>		>5.36 log ₁₀				✓	✓
<i>Clostridium difficile</i>			>5.43 log ₁₀		✓				✓	
Mycobactericidal	EN 14348:2005 (Phase 2, Step 1)	Quantitative suspension test	<i>Mycobacterium terrae</i> ATCC 15755	4 log ₁₀	>4 log ₁₀	Dr. Brill + Partner GMBH Laboratory, Germany	Clean	1 min	✓	✓
			<i>Mycobacterium avium</i> ATCC 15769						✓	✓
	EN 14563:2009 (Phase 2, Step 2)	Quantitative Carrier test	<i>Mycobacterium terrae</i> ATCC 15755	4 log ₁₀	>4 log ₁₀	Dr. Brill + Partner GMBH Laboratory, Germany	Clean	1 min	✓	✓
			<i>Mycobacterium avium</i> ATCC 15769						✓	✓
Virucidal	EN 14476:2013 (Phase 2, Step 1)	Quantitative suspension test	<i>Poliovirus</i> Type 1, LSc-2aba	4 log ₁₀	>5 log ₁₀	Dr. Brill + Partner GMBH Laboratory, Germany	Clean	1 min	✓	✓
			<i>Adenovirus</i> Type 5, strain Adenoid 75, ATCC VR-5		>6 log ₁₀				✓	✓
			<i>Murine Norovirus</i> Strain S99		>5 log ₁₀				✓	✓
	EN 14675:2015 (Phase 2, Step 1)	Quantitative suspension test	IPN 107TCID50 Salmonid fish viral infection	4 log ₁₀	>4 log ₁₀	Norwegian Veterinary Institute, Norway	Clean	5 min	✓	✓
					4 log ₁₀				>4 log ₁₀	Dirty
	EN 14675:2015 (Phase 2, Step 1)	Quantitative Carrier test	<i>Avian influenza virus</i> , (H10N7)	4log ₁₀	>4log	SVA	clean	1 min		✓
EN 14349:2007 (Phase 2, Step 2)	Quantitative Carrier test	PPV, strain 893/76	4log ₁₀	>5.3 log	SVA	Clean	5 min	✓	✓	
Fungicidal	EN 13624:2013 (Phase 2, Step 1)	Quantitative suspension test	<i>Candida albicans</i> ATCC 10231	4 log ₁₀	>4 log ₁₀	Dr. Brill + Partner GMBH Laboratory, Germany	Clean	1 min	✓	✓
			<i>Aspergillus brasiliensis</i> ATCC 16404						✓	✓
	EN 14562:2006 (Phase 2, Step 2)	Quantitative Carrier test	<i>Aspergillus brasiliensis</i> (niger) / black mold ATCC 16404	4 log ₁₀	>4 log ₁₀	Mikrolab Stockholm AB	Clean/Dirty	5 min	✓	✓
	EN 14562:2006 (Phase 2, Step 2)	Quantitative Carrier test	<i>Candida albicans</i> ATCC 10231	4 log ₁₀	>4 log ₁₀	Dr. Brill + Partner GMBH Laboratory, Germany	Clean	1 min	✓	✓
	EN 14562:2006 (Phase 2, Step 2)	Quantitative Carrier test	<i>Aspergillus brasiliensis</i> (niger) / black mold ATCC 16404	4 log ₁₀	>4 log ₁₀	Dr. Brill + Partner GMBH Laboratory, Germany and	Clean	2 min		✓
>4 log ₁₀					Dirty		2 min		✓	
Gram-Negative Bacteria	EN 13727:2014 (Phase 2, Step 1)	Quantitative suspension test	<i>Enterococcus hirae</i> ATCC 10541	5 log ₁₀	>5.11 log ₁₀	Dr. Brill + Partner GMBH Laboratory, Germany	Clean	1 min	✓	✓
	EN 14561:2006 (Phase 2, Step 2)	Quantitative Carrier test	<i>Enterococcus hirae</i> ATCC 10541	5 log ₁₀	>5.11 log ₁₀	Dr. Brill + Partner GMBH Laboratory, Germany	Clean	1 min	✓	✓
	EN 13727: 2015 (Phase 2, Step 2)	Quantitative suspension test	<i>Enterococcus hirae</i> ATCC 10541	5 log ₁₀	>5 log ₁₀	Food Safety Laboratory, Chung-Ang University, Seoul, South Korea	Dirty	2 min	✓	✓
	EN 13727:2014 (Phase 2, Step 1)	Quantitative suspension test	<i>Pseudomonas aeruginosa</i> ATCC 15442	5 log ₁₀	>5.11 log ₁₀	Dr. Brill + Partner GMBH Laboratory, Germany	Clean	1 min	✓	✓

	Test norm	Test type	Organism	Log reduction required	Log reduction achieved	Laboratory	Clean or dirty	Contact time	LC Std 400-200 PPM	LC Plus 650-400 PPM
Gram-Negative Bacteria	EN 14561:2006 (Phase 2, Step 2)	Carrier test	Pseudomonas aeruginosa ATCC 15442	5 log ₁₀	>5.11 log ₁₀	Dr. Brill + Partner GMBH Laboratory, Germany	Clean	1 min	✓	✓
	EN 13727:2015 (Phase 2, Step 2)	Quantitative suspension test	Pseudomonas aeruginosa ATCC 25922	5 log ₁₀	>5 log ₁₀	Food Safety Laboratory, Chung-Ang University, Seoul, South Korea	Dirty	2 min	✓	✓
	EN 13727:2015 (Phase 2, Step 2)	Quantitative suspension test ⁵	Escherichia coli ATCC 25922	5 log ₁₀	>5.76 log ₁₀	Internal test at Örebro University Hospital, Sweden	Clean/Dirty	2 min	✓	✓
	EN 13727:2015 (Phase 2, Step 2)	Quantitative suspension test	Campylobacter jejuni ATCC 33560	5 log ₁₀	>5.76 log ₁₀	Internal test at Örebro University Hospital, Sweden	Clean/Dirty	2 min	✓	✓
	EN 13727:2015 (Phase 2, Step 2)	Quantitative suspension test	Salmonella typhimurium ATCC 14028	5 log ₁₀	>5.76 log ₁₀	Internal test at Örebro University Hospital, Sweden	Clean	2 min	✓	✓
				5 log ₁₀	>5.76 log ₁₀		Dirty	2 min	✓	✓
			Legionella pneumophila ATCC 33152	5 log ₁₀	>5.76 log ₁₀		Clean	2 min	✓	✓
				5 log ₁₀	>5 log ₁₀		Dirty	2 min	✓	✓
	EN 14349:2012 (Phase 2, Step 2)	Quantitative suspension test	Aeromonas salmonicida ATCC 14174	5 log ₁₀	>7 log ₁₀	Internal test at Örebro University Hospital, Sweden	Clean	2 min	✓	✓
					>6 log ₁₀		Dirty	2 min	✓	✓
			Yersinia ruckeri ATCC 29473		>7 log ₁₀		Clean	2 min	✓	✓
					>5 log ₁₀		Dirty	2 min	✓	✓
	EN 13727:2015 (Phase 2, Step 2)	Quantitative suspension test	Klebsiella pneumoniae (ESBL) CCUG 54718	5 log ₁₀	> 5.76 log ₁₀	Internal test at Örebro University Hospital	Clean/Dirty	2min	✓	✓
			Enterococcus faecium (VRE) CCUG 56431						✓	✓
Acinetobacter baumannii (Clinical strain)			✓						✓	
Gram-Positive Bacteria	EN 13727:2014 (Phase 2, Step 1)	Suspension test	Staphylococcus aureus ATCC 6538	5 log ₁₀	>5 log ₁₀	Dr. Brill + Partner GMBH Laboratory, Germany	Clean	1 min	✓	✓
	EN 13727:2015 (Phase 2, Step 2)	Suspension test	Staphylococcus aureus ATCC 6538	5 log ₁₀	>5 log ₁₀	Food Safety Laboratory, Chung-Ang University, Seoul, South Korea	Dirty	2 min	✓	✓
	EN 14561:2006 (Phase 2, Step 2)	Carrier test	Staphylococcus aureus ATCC 6538	5 log ₁₀	>5 log ₁₀	Dr. Brill + Partner GMBH Laboratory, Germany	Clean	1 min	✓	✓
	EN 13727:2015 (Phase 2, Step 2)	Quantitative suspension test	Listeria monocytogenes CCUG 51681	5 log ₁₀	>5.76 log ₁₀	Internal test at Örebro University Hospital	Clean/dirty	2 min	✓	✓
		Quantitative suspension test	Streptococcus equi CCUG 37782	5 log ₁₀	>7 log ₁₀	Internal test at Örebro University Hospital	Clean	2 min	✓	✓
	EN 14349:2012 (Phase 2, Step 2)	Quantitative suspension test	Carnobacterium piscicola ATCC 3586	4 log ₁₀	>7 log ₁₀		Internal test at Örebro University Hospital	Clean	2 min	✓
4 log ₁₀				>5 log ₁₀	Internal test at Örebro University Hospital	Dirty	2 min	✓	✓	
Parasites	N/A	In vitro sporulation method for Coccidia	Coccidia spp		N/A	VidiLab, Sweden	Dirty	N/A		✓
	Salmon lice	In vitro	<i>Lepeophtheirus salmonis</i>	N/A	N/A	ILAB, Norway	N/A	1 min	100 ppm	✓
	N/A	In Vitro	<i>Gyrodactylus salaris</i>	N/A	N/A	Norwegian Veterinary Institute, Norway	N/A	10 sec	✓	✓
	N/A	In Vitro	<i>Gyrodactylus salaris</i>	N/A	N/A	Norwegian Veterinary Institute, Norway	N/A	1 min	100 ppm	✓

¹Initial **bacterial** spores 10⁶

²Initial **bacterial** spores 10⁸

³Initial **bacterial** spores 10⁸ and different bacteria strain

⁴Initial **bacterial** spores 10⁸ and 200 PPM

⁵Bactericidal effect (Clean condition for 2 minutes and Dirty condition for 5 minutes).

Third party test partners



Region Örebro län
Universitetssjukhuset Örebro



Veterinærinstituttet
Norwegian Veterinary Institute

For more information, please contact:

Rahma Wehelie, PhD

Senior Scientific Officer
Phone +46 522 506 475
Mail: rahma.wehelie@lifeclean.se

LIFE  CLEAN
LifeClean International AB
Kärstrandvägen 124B
451 76 Uddevalla, SWEDEN
+46 (0)522 104 04
info@lifeclean.se
www.lifeclean.se